



Information- Bite Away Insect Bite Healer 1300 559 519


Bite away® is a patented medical device which was developed for the external treatment of insect bites and stings.

SMALL, COMPACT DEVICE WITH GREAT EFFECT

The bite away® is an electronic device for the external treatment of insect bites and stings such as mosquitoes, sand fly/midges, wasps, bees, horse fly, fleas, bedbugs and mites/chiggers as well as sting caused by stinging nettle and jellyfish. bite away® is designed to be effective on any insect bite or sting that has a histamine reaction.

Small, portable and easy to use bite away® can be taken everywhere. Used as directed, bite away® is safe for all the family, even pets. bite away® is safe for people with allergies, pregnant women and epileptics. Safe to use for people with pacemakers and heart conditions.

When applied immediately after sting the bite away® prevents or minimises the itching so that it becomes insignificant and prevents a subsequent swelling. When applied later, even after hours, the itching will be minimised immediately or will disappear. The swelling will go down more quickly.

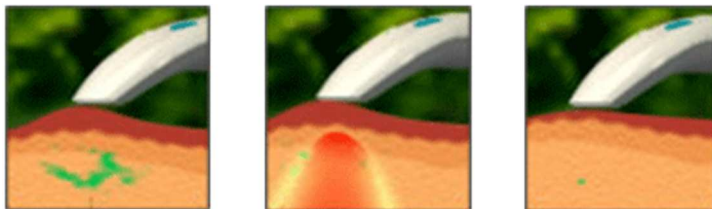


FAST, EFFECTIVE AND RELIABLE

The device works on the purely natural effect of concentrated heat. This effect takes place as a local thermal treatment at the puncture/bite site. With the device reaching a temperature of 50°C decomposing/denaturing the components on the insect poison and therefore preventing the histamine reaction. So the unpleasant consequences of an insect sting /bite can be immediately and permanently stopped.

TECHNOLOGY AND FUNCTION

The technology behind the bite away® is an integrated microchip. This is located directly on the round heating surface. Device operates on a voltage of 2.3V -3.6V and runs on 2 AA batteries. When the device flashes 3 times or has 3 acoustic tones this indicates that the batteries are low and need changing. The batteries are easily changed.

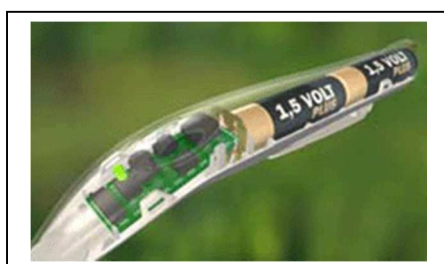

There are basically two treatment times selectable, left button for 3 seconds for children, and those with sensitive skin, and right button for 6 seconds for adults. After pressing the button the device will heat up. Once target temperature is reached, indicated by illumination of LED, treatment has started. Temperature will remain controlled and constant (49°C - 51°C) throughout treatment.

EASY TO USE



The device is best used slightly pressing onto the bite. The patient feeling a superficial warming of the skin and depending on the sensitivity of the person, there could be a slight response.

FACTS

- ✓ Quality German Technology, manufactured and tested in Germany
- ✓ Patented Medical Device
- ✓ TGA Approved (No. 223108)
- ✓ Dermatologically Tested
- ✓ Fully Insured underwriting by Lloyds of London
- ✓ 12 Months warranty
- ✓ Uses 2 AA batteries, lasting 300-500 applications, then easily replaced
- ✓ Clinically tested
- ✓ Splashproof, easy to use and portable
- ✓ Relieves swelling, itching and pain of insect bites and stings
- ✓ Fully endorsed by German Lifeguard Association (DLRG)
- ✓ Sold 200,000 in Europe in 2013

Bite Away Australia exclusive Australian Importers of Bite Away®- Insect Bite Healer.

SAFE FOR USE.....

On a worksite the bite away® device doesn't produce a spark when used.

Safe to use on all the family.

Safe for people with allergies, pregnancy, cancer, epilepsy and pacemakers or heart conditions.



FOR BEST USE.....

- Bite away® works best when slightly pressed into bite when treating bites/stings.
- For children show the child how to use device correctly and best if they self apply to skin.
- Do not use rechargables
- When treating blue bottle jelly fish stings you need to treat each welt until pain subsides.

DO NOT USE ON EYES, OPEN WOUNDS or MUCOUS MEMBRANES

For more information please go to www.biteawayaustralia.com.au

bite away Australia Pty Ltd

P O Box 1024, Byron Bay NSW 2481

www.biteawayaustralia.com.au